

Cabinet Installation Guide

Included with your cabinet


- ✓ 4 x M6 expansion bolts, to fix the cabinet to a wall
- ✓ 4 x M6 penny washers, to use inside the cabinet
- ✓ Entry code
- ✓ 2 x Heart Restarter side stickers
- ✓ Cabinet User Guide

Electrical Information


- This cabinet meets Class II double insulation standards
- The heater draws up to 50W and the LED strip draws 3W
- Hardwire the cabinet to a 4 Amp (time-lag) fused spur.

Mounting the cabinet to a building


1. Choose a site where the cabinet will not be under constant drainage (e.g. water dripping from a roof) and site which avoids the cabinet being in direct sunlight for long periods of time
2. Remove cabinet from packaging
3. Measure and mark the four fixing hole locations (W:340mm, H:275mm), ensuring the **bottom** of the cabinet is approximately 1.1m up off the ground.
4. Measure the mid-point between the 2 lower holes, then measure 50mm vertically up and mark. This is for the power cable hole. Offer up cabinet to check alignment

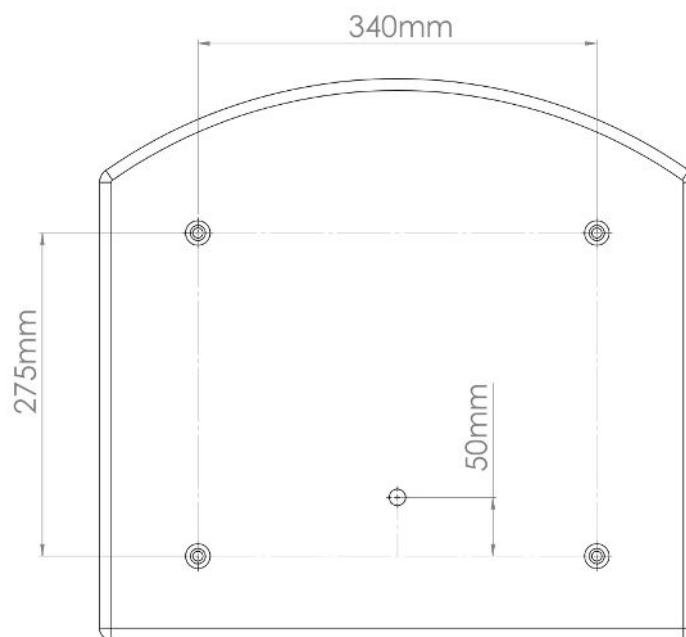

5. Drill out the four fixing holes, using an appropriate 12mm drill bit

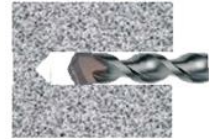

6. Drill out the power cable hole - angle the hole slightly uphill to prevent water ingress

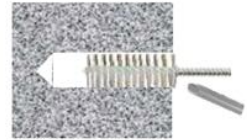


7. Insert the anchors and tighten the bolts to expand and secure the sleeves



8. Now unscrew the bolts, offer up the cabinet, and attach using the same bolts along with the M6 penny washers

Do not over-tighten as this may distort the cabinet if the mounting surface is not flat – use resin mounted bolts and spacers if necessary

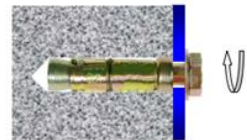

9. Run the power cable through the wall

10. A competent person should hard wire the cabinet to a fused spur, with isolation switch. The feed to the cabinet should also be protected by a Residual Current Device (RCD).

Final Checks

- Once there is power to the cabinet, the internal light will come on and the heater will be ready
- The heater is pre-set to come on at **8C** and turn off at **15C**
- Attach the two Heart Restarter stickers onto the sides of the cabinet (if desired).
- There is an Information Sticker inside the cabinet for maintenance instructions, electrical information and the cabinet serial number.

If you have any queries, please contact your cabinet supplier.